



Whisper mode: Turns off the word by word speaking to allow communication in quiet settings or of private messages.

Hold That Thought: This feature allows you to save messages for future use or when conversations change direction.

Search Feature: Search for words within the app's vocabulary. A purple outline will navigate you to the searched word. Searched words will be opened and accessible.

Edit: Touch edit to change icons, button background color, turn on/off link to secondary screens, add or edit buttons.

Open/Close: Allows you to close words to limit visual distraction and teach location of vocabulary. Open new words as language expands.

Babble: Toggle between the user's customized open/close configuration and ALL of the words. Allows language exploration and independent access to robust vocabulary.

Programming Lock: Use this to prevent curious little fingers from editing the vocabulary.

Settings: Change voices, back up and restore vocabulary settings, access the History Feature to monitor usage, and toggle between multiple users through multi-user functionality.

Message Window Sharing: Send the contents, of your message window as a text, email, or social media post.

